

Rigel Uni-Therm

The quickest and most accurate way to test all leading electrosurgical devices.

The Rigel Uni-Therm electrosurgical analyzer hosts a series of innovative features to enable a quick and accurate all-in-one solution for testing electrosurgical devices.

With a color screen and intuitive menu system, the Uni-Therm takes the complexity out of testing. On-board memory, test automation and a compact product footprint make it fast and convenient to use.

The Uni-Therm is capable of testing all modern and legacy electrosurgical devices, and features contact quality monitoring (CQM) analysis, high current power measurement up to 8A and high frequency leakage measurements with on-screen instructional diagrams to simplify the process.



Key Benefits

- Three self-contained resistive load banks (CQM, HF Leakage, Power)
- Tests all ESU's including those with high current vessel-sealing technology
- Meets all modern CQM test requirements and eliminates additional test equipment
- Be an expert in minutes with easy-to-follow on-screen instructions
- Speed up your testing by allowing the Uni-Therm to execute test sequences
- Eliminates the need to write down results, reducing errors and improving efficiencies
- Speed up testing, free up your time and improve safety

Electrical Test Functions

- High frequency/leakage
- High current load testing
- Peak-to-peak voltage
- Power Distribution
- Patient return plate alarm testing (CQM)

Uni-Therm Applications

- Routine testing of ESU generators
- Recalibration of ESU generators
- Production line testing
- Development tool for ESU R&D
- Type testing tool for ESU devices
- Evaluation tool for purchasing the correct ESU device
- Teaching tool for biomeds offering training on quality control procedures of ESU devices

Download your **FREE** guide to electrosurgery at **www.rigelmedical.com/guides**





► High current power measurement

Test all leading modern and legacy electrosurgical devices (ESU), with a maximum test current of 8A RMS for calibration of high current vessel sealing modes.





◀ High resolution, low induction load bank

With accurate high resolution load bank and the lowest possible inductance, the Uni-Therm offers an all-in-one solution for reliable power distribution measurements, without the need for external loads. Suitable for all ESU devices, the load bank ranges from 0-5115 Ω , in 5 Ω increments, to represent various tissue types.

► Simple user interface with detailed color screen

Cut testing times by following the easy navigation and step-by-step, on-screen color instructions and connection diagrams - including the simplification of leakage and power measurement test protocols.





Integrated automatic test protocol

Significantly reduce and simplify testing without the need for a PC or laptop connection. The Uni-Therm can also automate the activation of the ESU pedal or hand switch, from COAG to CUT.



Email: info@rigelmedical.com



Contact quality monitoring (CQM) analysis

An all-in-one solution with built-in analysis to test all contact quality monitoring systems in modern and legacy ESU devices. Simulate a fault to within 1Ω resolution.





◀ Small in size, small in price

With a footprint that's 50% smaller than competitors, the Uni-Therm ensures testing can be done even when physical space is at a minimum, making it easier to use, transport and store. Perfomance value is unequaled.

Meet the standards

Conducts all high frequency leakage tests, as per IEC 60601-2-2 requirements.









Technical Specifications

Power measurement True RMS value of applied

waveform

0 - 500W (RMS)

 $\pm (1W + 5\% \text{ of reading})$

100% up to 60 seconds 0 - 51150

5Ω

0 - 10kV (Peak to Peak) -

Closed load only ±(10% of reading + 15V)

Measurement is taken

between the active and

dispersive electrodes with

closed load only

0 - 700V (RMS)

 $\pm (10\% \text{ of reading} + 5V)$

0 - 6000mA with load bank

0 - 8000mA external load test

 \pm (2% of reading + 10mA)

1.4 - 20 (Vpeak / V RMS) The higher of the two peak

voltage measurements is

used for calculation

RMS Bandwidth

Power rating Accuracy

Duty cycle

Load bank

Resolution

Accuracy

Voltage

Accuracy

Accuracy Crest factor

Current (RMS)

Voltage (peak)

Instrumentation only 30 Hz to 10 MHz (-3 dB) With loads 30 Hz to 2.5 MHz (-3 dB)

Variable loads $5 - 5115\Omega$, steps @ 5Ω

(1023 steps)

 \pm (1%, + 0.5, -0.0 Ω of set load) Accuracy

Ceramic resistors (Non inductive)

Foot switch delay selectable Measurement delay

between 200 - 5000ms (10mSec resolution)

RF Leakage

Load array

(High Frequency Leakage)

Active From active part to earth From plate -receptacle -to earth Passive Load

Variable see power measurement

Fixed 2 x 200Ω ±1%, +0.5, -0.0 Ω

Accuracy

Contact Quality Monitoring (CQM)

Range 0 - 475Ω, steps @ 1Ω steps Motor driven potentiometer

 $\pm 5\% \pm 2 \Omega$

Accuracy

Alarm register High and low, manual confirmation

Ranging Manual or automatic

Output Connectors

Remote foot switch control (CUT) 2 x 4mm - yellow, single

relay contact

Remote foot switch control (COAG) 2 x 4mm - blue, single

relay contact

High Frequency leakage Through 4mm sockets and power measurements

PC download

0.5V/A, 100mA RF current Oscilloscope output

minimum input, un-calibrated,

Indication only

Rev 4.1

Isolation

10kV Isolation between measurement device and enclosure

Low Frequency Filter

100 Hz filter to avoid low-frequency disturbance or interference

General

Approx 5,000 records (4Mb) Memory Output CSV and SSS format

370mm x 300mm x 204 mm / 14.6" x 11.8" x 8" Dimensions

Weight 10kg / 22 lbs

10°C - 40°C / 50°F - 104°F Operating temperature 0°C - 50°C / 32°F - 122°F Storage temperature

Mains power 115/230V AC +10%; 48 to 66 Hz, 35 VA

Fuses 2 x 1.6 A (T) ceramic

Standard Accessories (supplied with Rigel Uni-Therm)

Mains lead USB lead

Calibration certificate

Optional Accessories

Med-eBase asset management software

CUT / COAG control interface cables

Bluetooth barcode scanner

Test lead set

Protective travel case (pelican case)

'An Introduction to Electrosurgery' guidance booklet

Service & Warranty

Uni-Therm comes with a free upgraded 24 month warranty

(subject to terms and conditions, available at www.rigelmedical.com/registerproduct)

Part Number

398A914 100V AC 398A912 120V AC

398A910 230V AC





Email: info@rigelmedical.com